
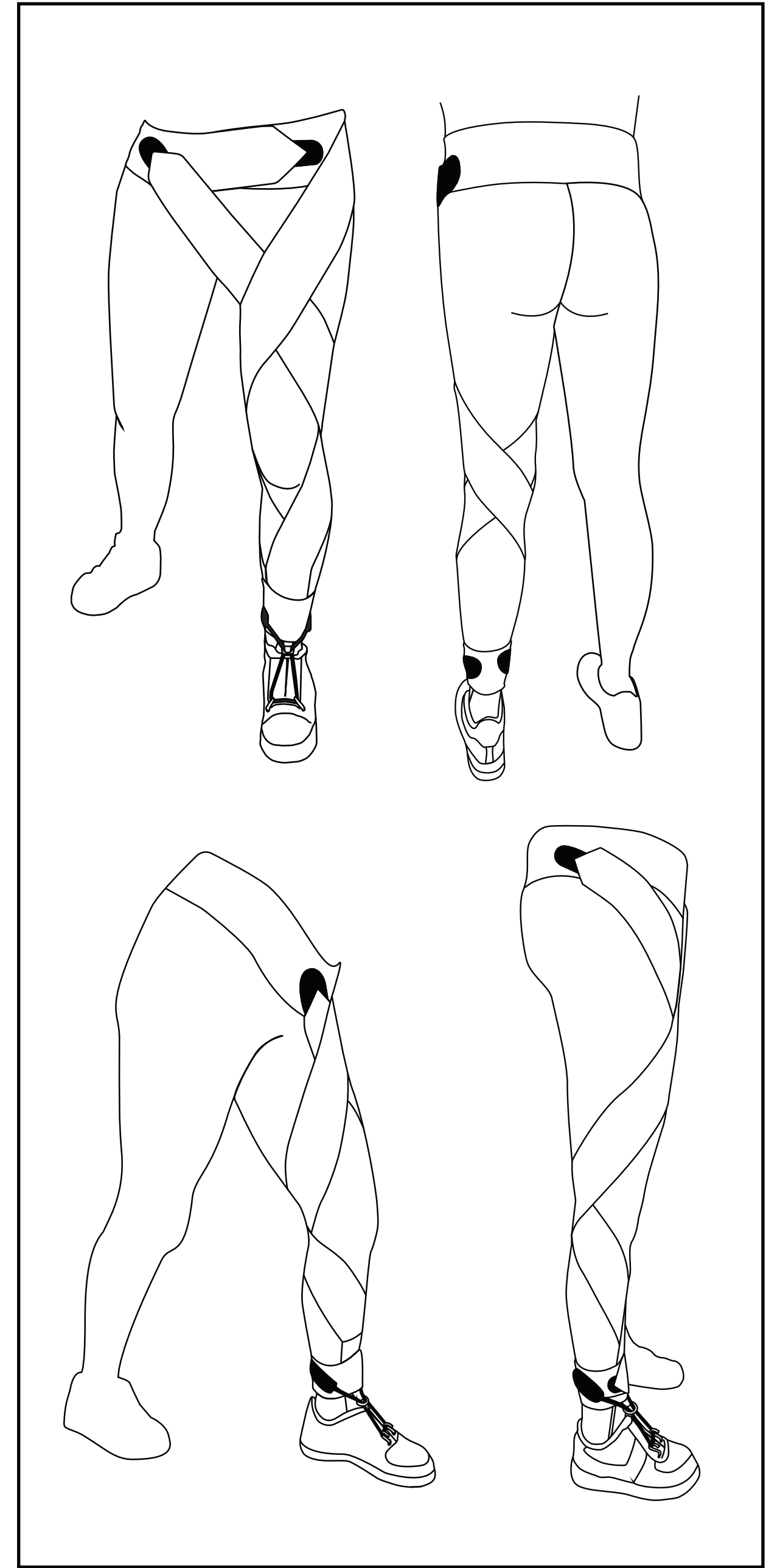
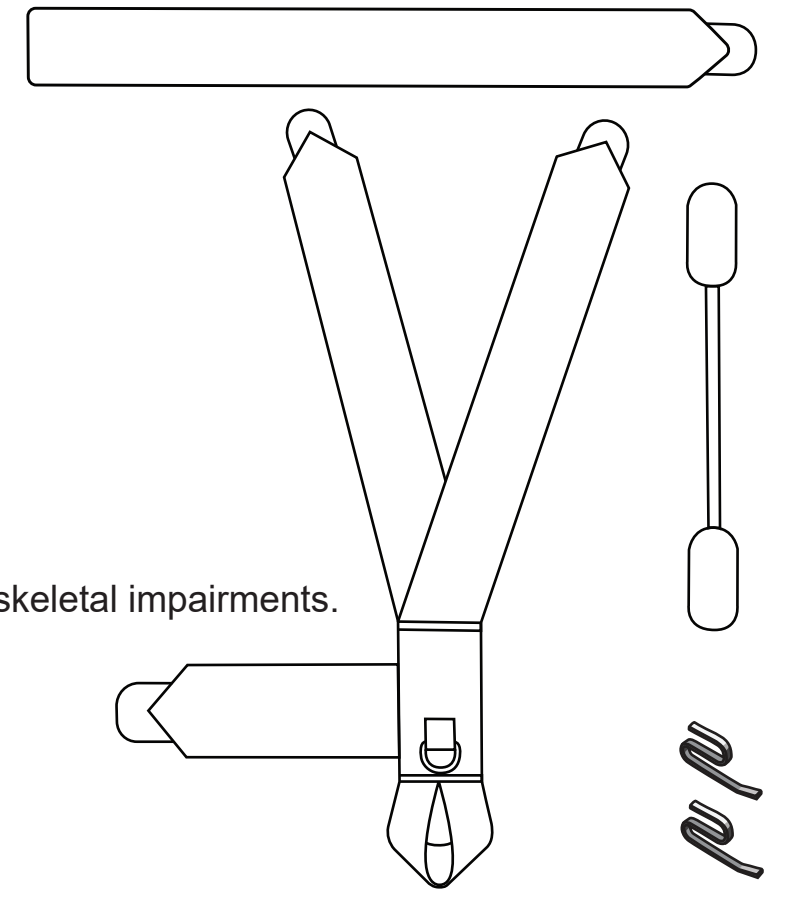
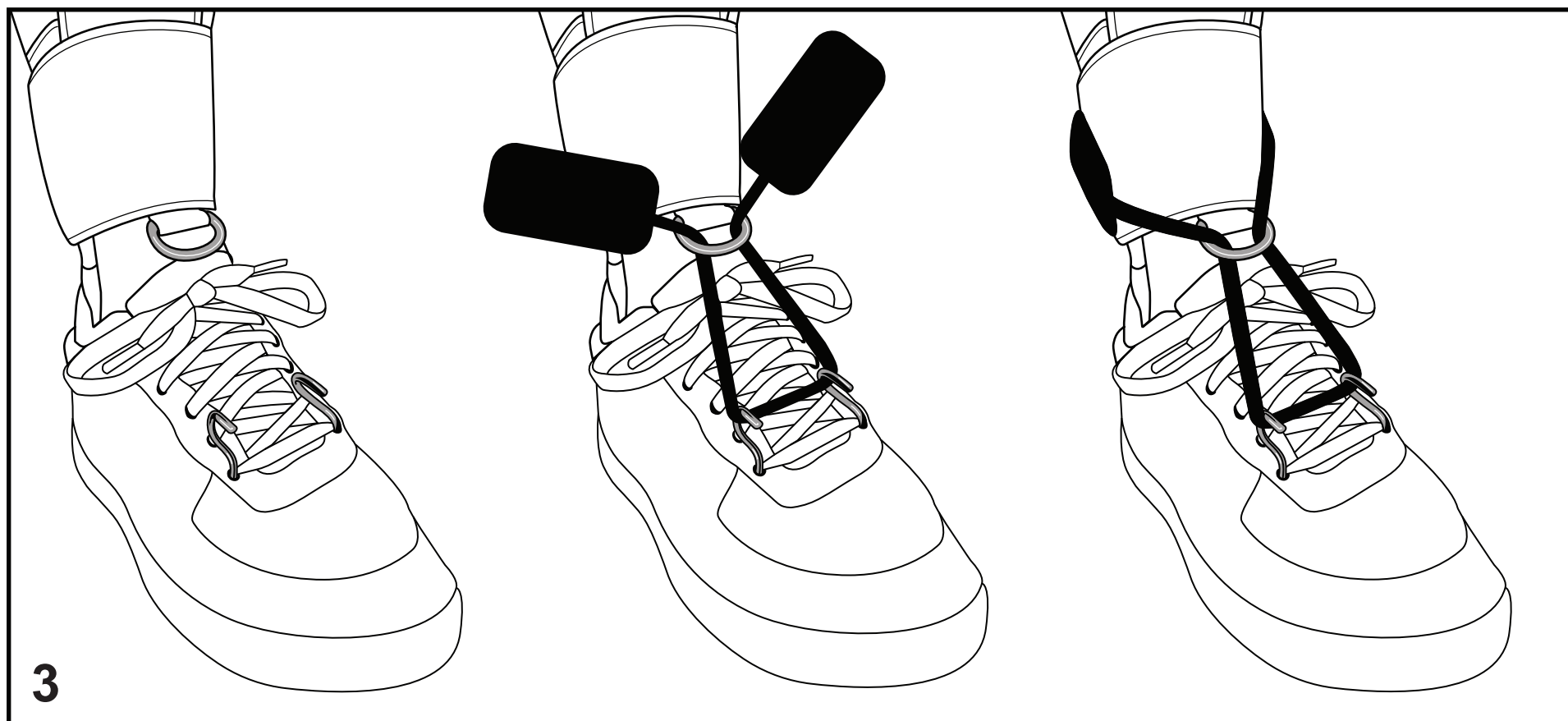
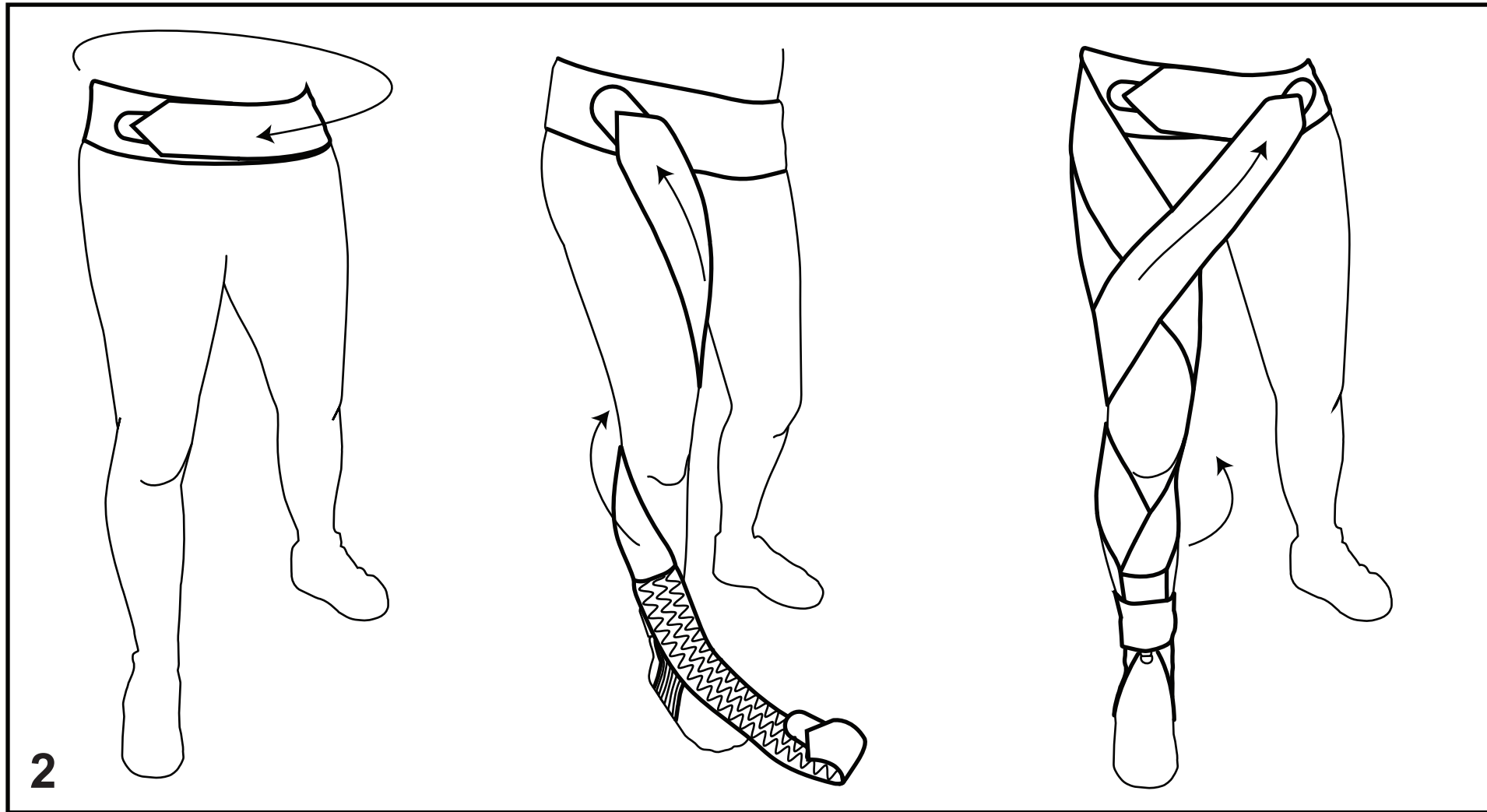


Flex4ward

Flex4ward is a fabric-based Class 1 medical device designed to assist lower limb movement and improve gait in individuals with neurological or musculoskeletal impairments. It assists hip and knee flexion, promotes ankle dorsiflexion, helps prevent knee hyperextension and enhances proprioception during walking.

Instructions for safe use:

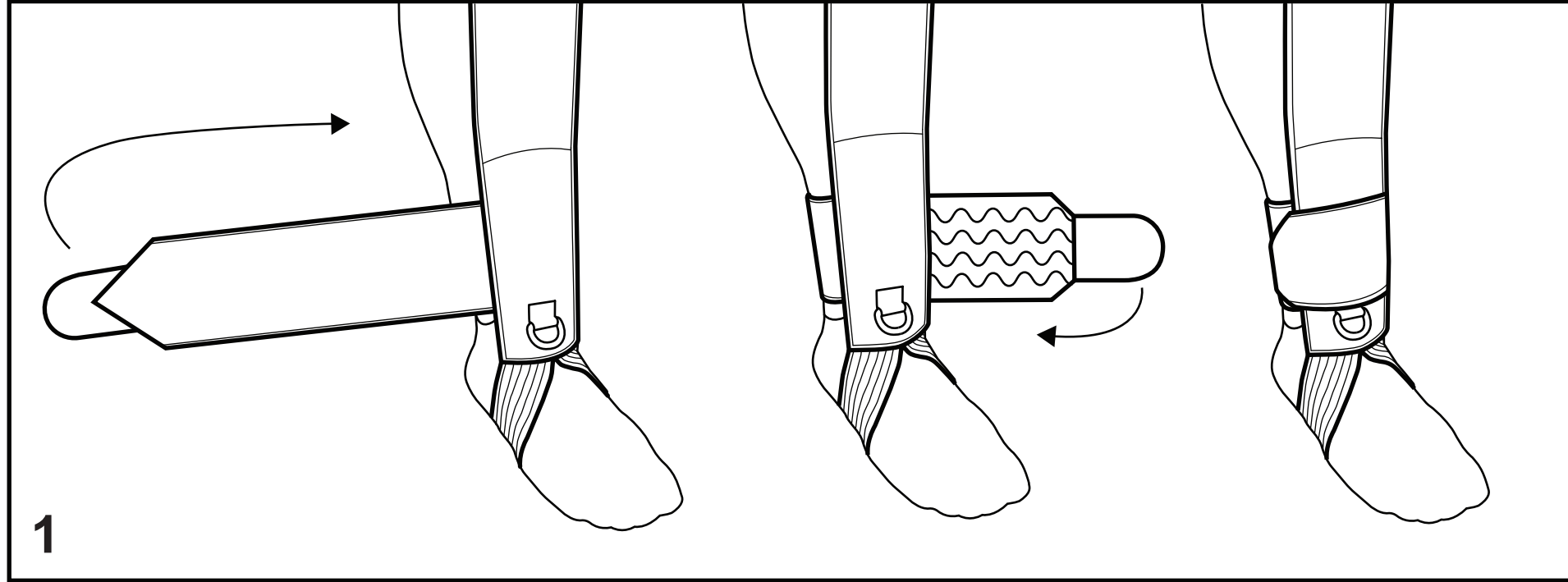

Flex4ward V.01

Care Instructions

- Hand wash in cold water with mild detergent.
- Air dry flat. Do not tumble dry or iron.
- Inspect regularly for wear and tear.

Warnings & Precautions

Use under supervision: Flex4ward should be fitted and used as directed. See above diagrams for fitting instructions. Scan the QR Code for instructional videos.

- **Do not use on broken or irritated skin:** Avoid application over wounds, rashes, or areas of active skin irritation.
- **Discontinue use if discomfort or adverse reaction occurs:** If you experience pain, numbness, tingling, or skin reaction, stop using the device and consult your clinician.
- **Check for wear and tear:** Inspect the device regularly. Do not use if damaged, frayed, or if fasteners are compromised.
- **Not suitable for unsupervised children:** Children should only use Flex4ward under adult supervision.
- **Do not overtighten:** Excessive tension may restrict circulation & cause discomfort or injury.
- **Do not modify the device:** Altering the design or fit may reduce effectiveness and compromise safety.
- **Keep away from heat sources:** Do not expose to open flames, high heat, or direct sunlight for prolonged periods.



Declaration of conformity reference:

This device complies with the applicable requirements of the UK Medical Devices Regulations 2002. A Declaration of Conformity has been issued by Propelus Ltd and is available upon request

Batch or serial number: Flex4ward V.01

Patent Pending UK

Registered Design UK

Propelus™ Ltd is a Registered Class 1 Medical Product

Propelus™ Ltd, South Church, Moray Street, Elgin. IV30 1JH. www.propelustd.com

Manufactured in Scotland, UK.

